

Instructions for use


Product name / Trade name

TOSI®-LumCheck

Product Description / Intended Purpose

The devices TOSI-LumCheck/LumCheck are used for routine monitoring and validation of cleaning efficacy of washer-disinfectors and ultrasonic cleaners. The devices are used by medical professionals and trained CSSD staff in CSSD departments of Hospitals and medical practices.

Instructions for use

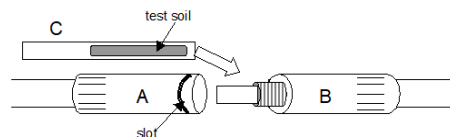

Gloves must be worn throughout the test procedure to avoid contamination of the test.

1. Unscrew LumCheck-device: detach part A (marked with slot) from part B (see picture).
2. Open protective pouch of TOSI®-LumCheck and insert the test object (part C) in part B as shown in picture. Do not touch the area covered with test soil.
3. Close LumCheck-device again.
4. Connect LumCheck-device with the rinse system of the washer (for example luer-lock) or reprocess it in the same way as done with cannulated instruments.
5. Open the device after the reprocessing cycle: disconnect part A from part B to remove the TOSI®-LumCheck without touching the test soil area! Check for visible residue.
6. In case of unsatisfactory results, please refer to the troubleshooting guide.



Pereg GmbH
Porschestraße 12
84478 Waldkraiburg
Germany



20100-25

Performance characteristics

Standardised test soil according to EN ISO 15883-5 consisting of purified bovine protein fractions.

In the event of malfunction / performance changes

Only completely clean instruments are safe. Residue left on a TOSI- LumCheck is an indication of an inefficient cleaning process.

Precautions:

- Open package carefully and handle with care.
- Don't eat and drink at work. No smoking. Keep away from food.
- Avoid contact with skin and eyes.
- On principle during handle with chemicals eye protection is necessary. Wear chemical resistant gloves.
- Don't use twice.
- Use only for the intended purpose.
- Read the instructions carefully before interpreting the results.

Restrictions on combinations

TOSI-LumCheck is only a test item to verify the cleaning performance of washer-disinfectors and ultrasonic cleaner.

Each test kit consists of:

25 X tests

Special storage and/or handling condition

- Store at +2°C to +25°C
- Store in closed original packaging.
- Store in original packaging. Do not open protective pouch until use.
- Store protected from light, heat and chemicals.
- Keep away from disinfectant and fumes.

Side-effects:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which you are located.

Safe disposal of device

Dispose of TOSI-LumCheck in accordance with valid local regulations.

Version and date of the issuance of the IFU.

Version: 2.0 Date: 22.07.2025