

Instructions for use for Endoswab

Special cleaning brushes for Flexible Endoscopes.
Available in sizes: 1.0 - 5.0 mm



1. EndoSwab cleaning brushes have a special cleaning head made out of cotton fibers. Cotton fibers absorb contamination (mucus and/or blood) and transport them from the internal channel when cleaning (see picture).
2. The correct size of Endoswab must be used (for sufficient friction, the channel should be of a little smaller diameter than the swab). Endoswab brushes must be pushed through the channel and removed at the other end. It should not be pulled backwards due to the absorbed contamination on the brush. In case of a heavily visible soil an additional swab should be used. This can be avoided by a proper pre-cleaning step.
3. In case of resistance when using the EndoSwab in a channel, do not attempt to push it through hard. Carefully pull the EndoSwab backwards and make sure the swab is still attached and intact.

Note: Faulty material/operation cannot be ruled out 100%. After use always check if the brush is still attached. EndoSwab brushes are single use products. If necessary Endoswab can be steam sterilised after placing it in a sterilizing pouch.

Storage: Store in the EndoSwab box in a clean place.

Content: 10 x single use Endoswabs

Marketing & Sales:

Produced For:
PEREG GMBH
Porschestr. 12
D-84478 Waldkraiburg
Germany