

What to do in case of blood residues?

- Possible reasons
- Monitoring
- Optimisation

A) Low cleaning efficiency

- ◆ Check detergent/dosing and water quality
- ◆ Check Machine/spray system
- ◆ Check cleaning parameters
- ◆ Check manual cleaning steps

⇒ PEREG clean monitoring and services

Visible blood residues or HemoCheck result



B) Problems with handling and reprocessing

- ◆ Check for overloaded machine
- ◆ Check water jet penetration through instrument trays
- ◆ Check if instrument are opened and dissembled
- ◆ Check manual reprocessing steps

⇒ Training of good reprocessing procedures

C) Special cleaning problems

- ◆ Check for contact with denaturing agents
- ◆ Check for heavily soiled instruments
- ◆ Check for long drying time
- ◆ Check for difficult to clean instruments

⇒ Pre-cleaning or improved cleaning might be necessary